



3

USE YOUR INHALER

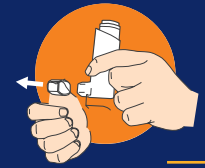

1. Remove the cap from the mouthpiece

2. Shake the inhaler well before each use

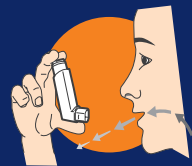

3. Hold the inhaler with the mouthpiece pointing towards you and breathe out as fully as you comfortably can through your mouth

4. Close your lips around the mouthpiece and tilt your head back, keeping your tongue below the mouthpiece. While breathing in deeply and slowly, press down on the centre of the dose indicator until the pressurised container stops moving in the actuator and a puff of medicine has been released. Then stop pressing the dose indicator.



5. When you have finished breathing in, remove the mouthpiece from your mouth. Hold your breath as long as you comfortably can, up to 10 seconds

6. Breath out gently



7. Repeat steps 2 to 6 to take the second puff



8. Replace the cap over the mouthpiece right away after use. Rinse your mouth with water to remove any excess medicine. Do not swallow.

Using the BEVESPI AEROSPHERE inhaler with a spacer


Your healthcare professional may prescribe a spacer with your inhaler to give you more time between releasing your medicine from the container and inhaling it. Make sure you read through the instructions that come with your spacer. When you are ready to take BEVESPI, shake the inhaler well and immediately attach the spacer to the mouthpiece of the inhaler. Follow the instructions to take BEVESPI as you normally would.

4

CLEAN & STORE


Clean the inhaler once each week after an evening dose has been taken. It is very important to keep your inhaler clean so that medicine will not build-up and block the spray through the mouthpiece.

Take the container out of the actuator. Do not clean the container or let it get wet.

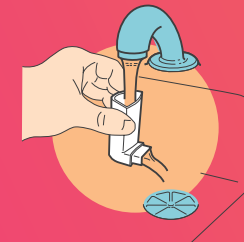


Take the cap off the mouthpiece. **Hold the actuator under the tap and run warm water through it for about 30 seconds.** Turn the actuator upside down and run warm water through it again for about 30 seconds.

Shake off as much water from the actuator as you can. Look into the actuator and the mouthpiece to make sure any medicine build-up has been completely washed away. Repeat washing and shaking off the water, until the actuator is clean. Let the actuator air-dry completely, such as overnight. **Do not** put the container back into the actuator if it is still wet.

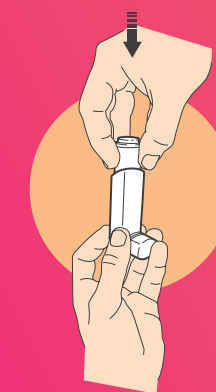

When the actuator is dry, gently press the container down in the actuator. Make sure you're not pressing down too hard on the container. You want to make sure it's in but don't want to cause a puff of medicine to be released. **Re-prime your inhaler after each cleaning.**

Make sure you store your inhaler in a dry place. Keep out of the sight and reach of children.

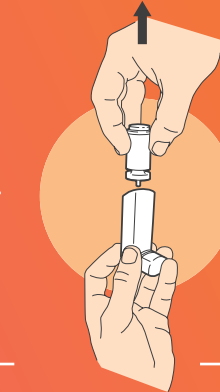

For patients prescribed



BEVESPI
AEROSPHERE®
(glycopyrronium/formoterol)
Inhalation Aerosol

USER GUIDE

GET TO KNOW YOUR INHALER

This guide has been created to show you how to use your Bevespi Aerosphere device. Please refer to the patient information leaflet that came with your inhaler for detailed information about your medicine. Talk to your doctor, nurse or pharmacist if you have any questions about the use of your inhaler.

If you get any side effects, talk to your doctor, pharmacist or nurse. This includes any possible side effects not listed in the package leaflet. You can also report side effects directly via the Yellow Card Scheme at <https://yellowcard.mhra.gov.uk>. By reporting side effects, you can help provide more information on the safety of this medicine.



Bevespi Aerosphere comes as a pressurised container with a dose indicator that fits into an actuator.

- Do not use the Bevespi Aerosphere actuator with a container from any other medicine.
- Do not use the Bevespi Aerosphere pressurised container with an actuator from any other inhaler.
- Do not use the inhaler if the drying agent has leaked out of its packet.
- Do not break, puncture, expose to high temperatures (above 50°C) or burn the pressurised container or let it get wet.
- Do not store Bevespi Aerosphere above 30°C.

Take your inhaler:

- 2** puffs in the morning. 
- 2** puffs in the evening. 

Stop using this medicine and seek medical help immediately if you get tightness of the chest, coughing, wheezing or breathlessness immediately after using Bevespi Aerosphere.

- Always take this medicine exactly as your doctor has told you.
- Do not use this medicine to treat a sudden attack of breathlessness or wheezing.



Do not use your inhaler when the arrow reaches "0". Even though your inhaler may not feel empty, you will not get the correct amount of medicine if you keep using it.

1 PREPARE FOR USE

After you take your inhaler out of the box, remove your inhaler from the foil pouch and throw away the pouch and drying packet. Write the date the pouch is opened on the inhaler label in the space provided. The inhaler can be used for up to 3 months after first opening the pouch. Do not use this medicine after the expiry date which is stated on the carton, pouch and pressurised container.

Before you use BEVESPI AEROSPHERE (glycopyrronium/formoterol fumarate) for the first time, make sure the pointer on the dose indicator is pointing to the right of the "120" inhalation mark in the dose indicator display window.

The dose indicator display window counts down in 10s. The number in the dose indicator display window will change after 20 puffs.

Refill your prescription when there are 20 puffs left.

2 PRIME FOR USE

Once you are ready to use your inhaler for the first time, you must prime it.

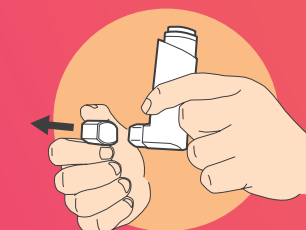


Figure 1

Hold the inhaler in the upright position away from your face **and shake it well.**

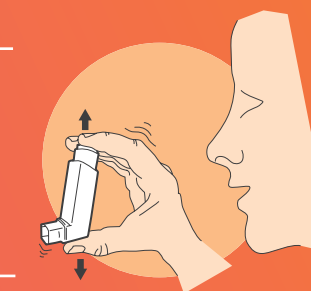


Figure 2

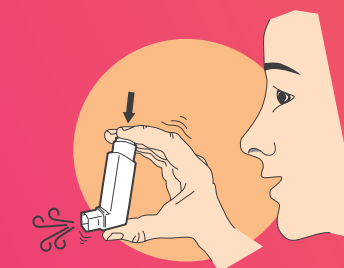


Figure 3

Press down firmly on the center of the dose indicator until the container stops moving in the actuator, to release a puff of medicine from the mouthpiece. You may hear a soft click from the dose indicator as it counts down during use.

Repeat the priming steps 3 more times (see Figure 2 and Figure 3). Shake the inhaler well before each priming puff.

After priming 4 times, **the dose indicator should be pointing to the right of "120"** and your inhaler is now ready to use.

Replace the cap until you are ready to use your inhaler.

If you haven't used your inhaler for more than 7 days, if the inhaler is exposed to low temperatures if you dropped it by accident, or after washing make sure you re-prime it before using it. To re-prime the inhaler, shake the inhaler well and press down on the center of the dose indicator twice to release a total of 2 puffs into the air away from your face.